

# Q50R

Store anywhere, Go everywhere.



# LET YOUR JOURNEY UNFOLD WITH THE Q50 R.

Fed up of clunky powerchairs that take up all the room? Say hello to the **QUICKIE Q50 R**. The premium, space-saving powerchair that folds easily in seconds to the size of a suitcase - ideal for storing neatly away until you get the urge for adventure!

And when you are ready to get out-and-about, the Q50 R packs a performance-punch to knock other folding powerchairs for six. From a dash around the shops to lengthy countryside exploring, go everywhere you want and more with a whopping **50 km range**, dual suspension and energy-efficient motors. Plus, the Q50 R is solidly built upon **40+ years** of powerchair experience, so you can be confident that it'll keep you safe.

Let's find out how the Q50 R fits into your everyday life.



Scan the QR code with your phone's camera to watch video.

## STORE IN THE SMALLEST SPACES!

When folded, the Q50 R is so compact it can be easily stored away in the porch, underneath the stairs or even in the boot of a car. Thanks to its rear anti-tips, it's easily moveable when standing upwards too.



# SIMPLY EFFORTLESS FOLDING.

There's no troublesome mechanisms - just flip-up the clip at the back, fold the footplate and push the backrest downwards to fold the Q50 R straight to the floor. You don't even need to remove the battery. How simple is that?



## IDEAL FOR CAR HOISTS

Planning a road trip? At **37.5kg**, the Q50 R is ideal for hoisting in-and-out of your car. You can even remove the battery to further reduce its weight down to just **32 kg**.



Scan the QR code with your phone's camera to watch video.

# INDOOR MANOUEVERABILITY

It might store out of the way, but you'll want this powerchair with you all day. Whether it's at home, a bustling shopping centre or crowded public transport, the Q50 R's compact size and precision control gets you out of any tight situation.



## NARROW WIDTH

Worried about clipping the sides of your Q50 R around the home? Don't be - its slim **600 mm** base width can fit easily through narrow doorways and negotiate busy rooms.

## EASY TRANSFERS

Easily side transfer in-and-out of your Q50 R by flipping the armrests upwards. The controller also swings outwards, so you can get closer to tables too. Prefer to transfer standing? The centre-mount footrest conveniently flips-upwards, giving you all the space you need.



Scan the QR code with your phone's camera to watch video.



## 1220 mm TURNING CIRCLE

Unlike a mobility scooter, the Q50 R has a small 1220 mm turning circle, making it ideal for indoor environments. Effortlessly turn in narrow hallways, kitchens and other restricted spaces without the need for complicated manoeuvres.



## VSI CONTROLLER

To use a powerchair indoors, you need precision-movement. That's why the Q50 R comes with the VSI controller by the world-leading Curtiss-Wright Corporation. Intuitive to use, this established and reliable controller responds precisely to your commands. It also helps you keep track of remaining battery life too.



# OUTDOOR PERFORMANCE

---

**Places to go, people to see? The Q50 R gives you more than just space-saving ingenuity. It's also packed with the outdoor features to get you reliably from A to B. So when you want to leave the car at home, you can just un-fold and go.**

## **DUAL SUSPENSION**

The Q50 R always has a spring in its step thanks to the front rubber damper and rear shock absorbers. Providing responsive cushioning when driving over bumpy or uneven terrain, it'll reduce the amount of shock and vibration that passes through the powerchair. So you can sit-back, relax and enjoy a smoother ride.

### GREATER TRACTION

Get to grips with greater traction thanks to the large **12" drive wheels**. These wheels are designed for real outdoor performance, providing better grip on multiple surfaces. Rear mudguards also help to keep you clean from water or grit flicking up whilst driving.



### UPHILL STABILITY

The Q50 R's longer **1030 mm** length provides a greater level of stability, so you can safely travel up gradients of up-to 10 degrees. For additional safety, rear anti-tips are also supplied as standard too.

### INCREASED RIGIDITY

The Q50 R's durable aluminium frame uses oval tubing (as opposed to round). Stronger and less likely to flex, it reduces movement in the frame whilst travelling over uneven ground to provide a more comfortable ride. It's also covered by a **5 year** warranty.



# PREMIUM, PROVEN PERFORMANCE.

We've been designing powerchairs for over 40 years, working alongside world-class manufacturers to deliver solidly-built powerchairs that have the very-best driving performance, motors, range and reliability that others just can't match.

## 50 mm

### KERB CLIMBING

Supplied by Linix (one of the world's leading motor manufacturers), the **200W high efficiency motors** provide all the power you need to overcome tricky terrain (including climbing obstacles of up to 50 mm). Highly optimised, they'll also deliver the very-best range from your batteries too.



### LOW-RESISTANCE PUSHING

Pop the Q50 R into freewheel mode and its low resistance motors will work with you to make pushing the Q50 R effortless. The padded foam around the top of the backrest bar also provides a more comfortable grip to push from too.



# 50

## KM RANGE

Thanks to our optimised motor efficiency, the Q50 R can deliver a whopping 50 km (31 miles) range\* from its Lithium Ion battery (and that's from a single charge too).

### CHARGE EVEN WHEN FOLDED!

Because you don't need to remove the battery when folding your Q50 R, it can also be conveniently charged via the socket on the controller.



### REMOVABLE, LITHIUM-ION BATTERY

Weighing just **5.5kg**, and with its own dedicated charging socket, there's no need to un-pack your Q50 R when running low on charge. Just pop-off the battery and charge separately! It's also removable when folded too.



Scan the QR code with your phone's camera to watch video.

\* Range is calculated to ISO standard 7176 part 4 wheelchair energy consumption theoretical range.

# MAKE YOURSELF COMFORTABLE.

Wherever you go, whatever you do, it'll always be in premium comfort thanks to the Q50 R's tension-adjustable seat.




Scan the QR code with your phone's camera to watch video.

450 mm  
550 mm

## HEIGHT-ADJUSTABLE BACKREST

Need higher back support? Simply adjust the backrest between 450 and 550 mm by using the two screws at the back. The foam backrest cover also has the same breathable fabric to keep you cool and dry throughout the day.

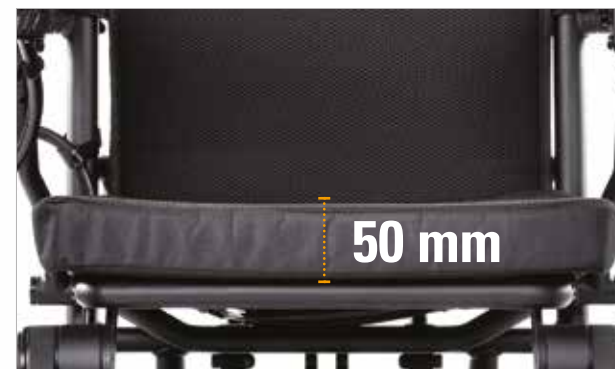

## BREATHABLE FOAM CUSHION

The foam cushion doesn't just keep you comfortable, it also has a breathable cover that allows body heat to pass through, keeping you cool and dry on those longer trips.



## ADJUSTABLE SEAT SLING

Unlike a solid seat base that you'll find on most folding powerchairs, the Q50 R's seat (and back) have tension adjustable straps. These straps provide extra-give when negotiating bumpy terrain and can be fine-tuned to suit your shape. They also provide better airflow and ventilation as your body's heat can pass completely through.

# THE FINISHING TOUCH.

With its elegant matt-black frame, the Q50 R is a real centrefold. So why don't you really get heads turning with a selection of 3 accent colours?

GREEN

BLUE

RED

Now that you've read all about the Q50 R, why not experience it for yourself?



Scan the QR code with your phone's camera to arrange your free test-drive.

## STORAGE POUCHES

Need somewhere to put that shopping? The Q50 R has two spacious, zippered storage pouches on the back of the backrest upholstery and underneath the seat.



# TECHNICAL DATA

<b>Maximum user weight</b>	136 kg	<b>Maximum speed</b>	6 km/h
<b>Folded width</b>	600 mm	<b>Batteries</b>	30 Ah, Lithium Ion
<b>Folded length</b>	800 mm	<b>Electronics</b>	VSI (Curtiss & Wright)
<b>Folded height</b>	400 mm	<b>Charger</b>	24 V, 5 Amp
<b>Total width</b>	600 mm	<b>Motors</b>	2 x 200 Watt
<b>Total length</b>	855-1030 mm	<b>Maximum safe slope</b>	10 Degrees (dynamic stability)
<b>Total height</b>	905-1005 mm	<b>Wheels</b>	12" Solid (rear) / 8" Solid (front)
<b>Backrest height</b>	450-550 mm	<b>Accent colours</b>	Red, green and blue
<b>Seat height</b>	490 mm (without cushion)	<b>Total weight</b>	37.5 kg (including battery)
<b>Seat width</b>	430 mm (cushion width)	<b>Battery weight</b>	5.5 kg
<b>Seat depth</b>	450 mm (cushion depth)	<b>Weight without battery</b>	32 kg
<b>Armrest height</b>	280 mm (from the base of the seat)		
<b>Range</b>	50 km (31 miles)*		

Available on:



Sunrise Medical cares about your safety. Please read your owner's manual for safe operating instructions and maintenance guidelines.

\* Range is calculated to ISO standard 7176 part 4 wheelchair energy consumption theoretical range.

In line with our policy of continual product improvement, Sunrise Medical reserves the right to make changes at any time, and without notice. For the latest information, please visit [www.sunrisemedical.co.uk](http://www.sunrisemedical.co.uk)

If you are visually impaired, this document can be viewed in PDF format at [www.sunrisemedical.co.uk](http://www.sunrisemedical.co.uk)



**Sunrise Medical Limited**

Thorns Road, Brierley Hill, West Midlands, DY5 2LD

Tel.: +44 (0) 845 605 66 88

Fax: +44 (0) 845 605 66 89

E-Mail: [enquires@sunmed.co.uk](mailto:enquires@sunmed.co.uk)

[www.sunrisemedical.co.uk](http://www.sunrisemedical.co.uk)